

Liquid Foil

Intended Purpose

Separating solution to facilitate the separation of acrylic from stone model

Other Materials / Equipment Needed

For Liquid

1. Brush

For Powder

1. Scale or measuring spoon
2. 4-oz. bottle with cap

Instructions for Use

For Liquid

1. Soak stone model in water until saturated.
2. Remove from water. Dry surface.
3. Paint thin layer uniformly over area of stone model that will come in contact with acrylic.
4. Let dry approximately three minutes.

For Powder

1. Add 1.5 grams (approximately 1/2 teaspoon) of powder to 70 mL of warm water.
2. Shake vigorously for several minutes and let liquid sit until powder is fully dissolved.
3. Continue with Instructions "For Liquid" above.

Handling Precautions

Keep material in original container tightly closed in upright position. Store in a well-ventilated place. Keep cool. Handle in accordance with good industrial hygiene and safety practice. Use only in well-ventilated areas. Avoid contact with skin, eyes or clothing. Avoid breathing dust. Wash thoroughly with soap and water after handling. Wash contaminated clothing before reuse. Minimize dust generation and accumulation.


Disposal

Dispose of contents and container in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase only from an authorized reseller. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

Lang Dental Mfg. Co., Inc.
175 Messner Dr
Wheeling, IL 60090 USA
www.LangDental.com
Made in USA
Ph +1 847-215-6622
Fax +1 847-215-6678

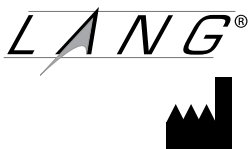

© 2021 Lang Dental Mfg. Co., Inc. LANG is a registered trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 22 R. 9