

# Ortho-Jet™ Tinting Concentrate

## Intended Purpose

Addition of color to Ortho-Jet™ clear liquid for fabrication of orthodontic appliances by a dental professional

## Other Materials / Equipment Needed

1. Glass dropper
2. Ortho-Jet™ liquid - clear

## Instructions for Use



For consistent results always shake liquid before use, as catalyst may have separated from monomer.

1. Add a small amount of Ortho-Jet™ Tinting Concentrate to Ortho-Jet™ clear liquid.
2. Stir until homogenous.
3. Repeat until desired tint is achieved.

## Handling Precautions

Contains methyl methacrylate

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surface. – No smoking.

Causes skin irritation. May cause an allergic skin reaction.

May cause respiratory irritation.

Wear protective gloves/protective clothing/eye protection/face protection.

Keep material in original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

## Disposal

Dispose of contents and container in accordance with local, regional and national regulations.

## Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase only from an authorized reseller. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



MediMark® Europe. 11 rue E. Zola  
38100 Grenoble. France

Lang Dental Mfg. Co., Inc.  
175 Messner Dr  
Wheeling, IL 60090 USA  
[www.LangDental.com](http://www.LangDental.com)  
Made in USA  
Ph +1 847-215-6622  
Fax +1 847-215-6678



Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2021 Lang Dental Mfg. Co., Inc. LANG is the registered trademark of Lang Dental Mfg. Co., Inc. ORTHO-JET is the trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 33 R. 3 2021-05