

Acrylic Primer

Product Identification

Bonding enhancer

Package Contents

REF 1602	Net Vol. 30ml, brushes
REF 1607	Net Vol. 3.8L
REF 1608	Net Vol. 19L

Intended Purpose

This product is intended to be used as an acrylic bonding agent.

Storage Conditions/Instructions

Close container tightly after use. Store in original container in a cool, dry, well-ventilated location, away from sources of ignition and light.



F - Highly Flammable

Operating/Handling Instructions

Liquid may cause skin irritation. Avoid contact with skin, eyes and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation. Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at www.langdental.com or by contacting manufacturer.



Xi - Irritant

Warning: flammable liquid. Contains: Methyl Methacrylate Monomer, stabilized. Flash Point: 52.7°F (11.5°C), NFPA, NPCA-HMIS #: H-2, F-3, R-2.

Contraindications

Acrylic Primer is contraindicated for persons with allergies to methyl methacrylate monomer.

Other Materials/Equipment Needed

1. Brush

Instructions for Use

1. Paint Acrylic Primer liquid over original acrylic surface to which new acrylic will be added.
2. Allow Acrylic Primer to dry completely.
3. Add new acrylic.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

EC	REP
----	-----

MediMark[®] Europe
BP 2332
38033 Grenoble Cedex 2
France



Lang Dental Mfg. Co., Inc.
Wheeling, IL 60090-0969
www.LangDental.com
Made in USA
Ph 847-215-6622
Fax 847-215-6678

The Lang logo features the word 'LANG' in a bold, italicized, sans-serif font. A thin horizontal line is positioned above the letters. A diagonal line starts from the bottom left and points towards the 'L'.