

Acrylic Primer

Product Identification

Bonding enhancer

Package Contents

REF 1602 Net Vol. 30ml, brushes
REF 1607 Net Vol. 3.8L
REF 1608 Net Vol. 19L

Intended Purpose

This product is intended to be used as an acrylic bonding agent. When used as instructed, product is 100% effective.

Storage Conditions/Instructions

Close container tightly after use. Store in original container in a cool, dry, well-ventilated location, away from sources of ignition and light.



F - Highly Flammable



Xi - Irritant

Operating/Handling Instructions

Liquid may cause skin irritation. Avoid contact with skin, eyes and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation. Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at www.langdental.com or by contacting manufacturer.

Warning: flammable liquid. Contains: Methyl Methacrylate Monomer, stabilized. Flash Point: 52.7°F (11.5°C), NFPA, NPCA-HMIS #: H-2, F-3, R-2.

Contraindications

Acrylic Primer is contraindicated for persons with allergies to chemicals.

Other Materials/Equipment Needed

1. Brush

Instructions for Use

1. Paint Acrylic Primer liquid over original acrylic surface to which new acrylic will be added.
2. Allow Acrylic Primer to dry completely.
3. Add new acrylic.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty


Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

Authorized Representative in EU:

MediMark[®] Europe
BP 2332
38033 Grenoble Cedex 2
France



Lang Dental Mfg. Co., Inc.
Wheeling, IL 60090-0969
www.LangDental.com
Made in USA
Ph 847-215-6622
Fax 847-215-6678

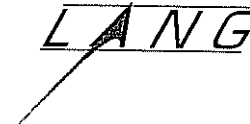

LDD 16 REV.3