

Contemporary Ortho-Jet

Product Identification

Fast curing orthodontic acrylic resin

Package Contents

REF 1520	Net Wt. 100g
REF 1530	Net Wt. 454g
REF 1550	Net Wt. 2.3kg
REF 1570	Net Wt. 11.3kg
REF 1580	Net Wt. 45.4kg
REF 1503	Net Vol. 118ml
REF 1504	Net Vol. 236ml
REF 1506	Net Vol. 946ml
REF 1507	Net Vol. 3.8L
REF 1528	Net Vol. 6x30ml; Net Wt. 100g; glass dropper; powder dispenser cap
REF 1593	Net Wt. 6x45g; Net Vol. 118ml; glass dropper

Intended Purpose

This product is intended for the fabrication of orthodontic appliances.

Storage Conditions/Instructions

Close container tightly after use. Store in original container in a cool, dry, well-ventilated location, away from sources of ignition and light.



F - Highly Flammable

Operating/Handling Instructions

Liquid may cause skin irritation. Avoid contact with skin, eyes and clothing.

Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation.

Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at www.langdental.com or by contacting manufacturer.



Xn - Harmful

Operating/Handling Instructions (Continued)

Warning: flammable liquid. Contains: Methyl Methacrylate Monomer, Stabilized. Flash Point: 52.7°F (11.5°C), NFPA, NPCA-HMIS #: H-2,F-3,R-2.

Contraindications

Contemporary Ortho-Jet is contraindicated in patients with allergies to methyl methacrylate.

Other Materials/Equipment Needed

1. Orthodontic wire
2. Sticky wax
3. Liquid Foil (Tin foil substitute solution)
4. Arbor band or stone
5. Jet Seal (Acrylic sealant) or pumice and high shine
6. Aquapres™ (Pressure curing unit), optional

Instructions for Use

Use Contemporary Ortho-Jet Powder with clear Ortho-Jet Liquid. Use Contemporary Ortho-Jet Liquid with clear Ortho-Jet Powder. For consistent results, always shake Ortho-Jet liquid before use, as catalyst may have separated from monomer.

1. On a prepared model, bend wire in conventional manner.
2. Attach wire to model with sticky wax.
3. Place model in water and soak for five minutes.
4. Remove model from water and blow off excess water. Paint a thin coat of Liquid Foil on model and let dry for about one minute.
5. Moisten surface with Ortho-Jet liquid. Then apply Ortho-Jet powder from dispenser bottle. Alternate liquid and powder until desired shape and thickness is achieved. Always begin and end with liquid for best results.
6. Allow acrylic to gel for approximately two minutes. Cure acrylic, while still on model an additional 10 minutes, either on bench or for better results, in Aquapres at 30 psi.

Instructions for Use (Continued)

7. Remove from model after acrylic is cured and trim using an arbor band or stone. Grind to a smooth finish.

8. Polish using Jet Seal or pumice and high shine.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

EC REP

MediMark® Europe
BP 2332
38033 Grenoble Cedex 2
France

Lang Dental Mfg. Co., Inc.
Wheeling, IL 60090-0969 USA
www.LangDental.com
Made in USA
Ph 847-215-6622
Fax 847-215-6678



LANG™



LDD 15 R.5