

# El Cor

## Product Identification

Orange solvent with lanolin

## Package Contents

REF 0404            236 ml

REF 0405            473 ml

REF 0406            946 ml

## Intended Purpose

This product is intended for removing zinc oxide eugenol cements and pastes. When used as instructed, product is 100% effective.

## Storage Conditions

Keep container closed. Store away from heat, sparks, and flame in cool, dry area in original container.

## Operating/Handling Instructions



Xi = Irritant

Liquid may cause skin irritation. Avoid contact with skin, eyes, and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation. Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at [www.langdental.com](http://www.langdental.com) or by contacting manufacturer.

**Warning:** Combustible liquid. Flash point 122°F (50°C), NFPA #: H-1, F-2, R-0.

## Contraindications

El Cor is contraindicated in users with allergies to orange oil or lanolin.

## **Other Materials/Equipment Needed**

1. Disposable cloth

## **Instructions for Use**

1. Apply small amount to disposable cloth.
2. Wipe until clean.

## **Disposal**

Dispose of unused material in accordance with local, state, and national legislation.

## **Warranty**

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is used for other than intended purpose or other than instructed.

### **Lang Dental Mfg. Co., Inc.**

Wheeling, IL 60090-0969

[www.langdental.com](http://www.langdental.com)

Made in USA

Ph 847-215-6622

Fax 847-215-6678

The logo for Lang Dental features the word "LANG" in a stylized, italicized, serif font. The letters are white with a black outline and are set against a black horizontal bar. A thin, light-colored diagonal line extends from the bottom left corner of the bar towards the bottom left of the page.