

El Cor

Product Identification

Orange solvent with lanolin

Package Contents

REF 0404 236 ml

REF 0405 473 ml

REF 0406 946 ml

Intended Purpose

This product is intended for removing zinc oxide eugenol cements and pastes. When used as instructed, product is 100% effective.

Storage Conditions

Keep container closed. Store away from heat, sparks, and flame in cool, dry area in original container.

Operating/Handling Instructions



Xi = Irritant

Liquid may cause skin irritation. Avoid contact with skin, eyes, and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation. Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at www.langdental.com or by contacting manufacturer.

Warning: Combustible liquid. Flash point 122°F (50°C), NFPA #: H-1, F-2, R-0.

Contraindications

El Cor is contraindicated in users with allergies to orange oil or lanolin.

Other Materials/Equipment Needed

1. Disposable cloth

Instructions for Use

1. Apply small amount to disposable cloth.
2. Wipe until clean.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is used for other than intended purpose or other than instructed.

Lang Dental Mfg. Co., Inc.
Wheeling, IL 60090-0969
www.langdental.com
Made in USA
Ph 847-215-6622
Fax 847-215-6678

