

Immediate

Product Identification

Very soft denture reline acrylic resin

Package Contents

REF 3720	Net Wt. 100g
REF 3730	Net Wt. 454g
REF 3750	Net Wt. 2.7kg
REF 3770	Net Wt. 11.3kg
REF 3780	Net Wt. 45.4kg
REF 3703	Net Vol. 118ml
REF 3704	Net Vol. 236ml
REF 3706	Net Vol. 946ml
REF 3707	Net Vol. 3.8L
REF 3708	Net Vol. 19L
REF 3723	Net Wt. 100g; Net Vol. 118ml; Acrylic Primer: Net Vol. 10ml; mixing cups; measuring vials, brushes
REF 3734	Net Wt. 454g; Net Vol. 236ml; Acrylic Primer: Net Vol. 10ml; mixing cups; measuring vials, brushes

Intended Purpose

This product is intended for the fabrication of denture relines. When used as instructed, product is 100% effective.

Storage Conditions/Instructions

Close container tightly after use. Store in original container in a cool, dry, well-ventilated location, away from sources of ignition and light.



F - Highly Flammable



Xn - Harmful

Operating/Handling Instructions

Liquid may cause skin irritation. Avoid contact with skin, eyes and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation. Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at www.langdental.com or by contacting manufacturer.

Operating/Handling Instructions (Continued)

Warning: flammable liquid. Contains: Methyl Methacrylate Monomer, Stabilized. Flash Point: 52.7°F (11.5°C), NFPA, NPCA-HMIS #: H-2,F-3, R-2.

Contraindications

Immediate is contraindicated in patients with allergies to chemicals.

Other Materials/Equipment Needed

1. Acrylic bur or ruby carver
2. Acrylic Primer (Acrylic bonding agent)
3. Brush
4. Spatula
5. Sharp knife or emery disc

Instructions for Use

For consistent results, always shake Immediate liquid before use, as catalyst may have separated from monomer.

1. Clean and disinfect denture.
2. Trim down any high spots and relieve denture if necessary with acrylic burs or ruby carvers.
3. Coat denture area to be relined with Acrylic Primer.
4. Protect any surfaces not to be relined.
5. Adding liquid first, mix a filled vial of Immediate liquid (smaller vial about 4ml) with a filled vial of Immediate powder (larger vial about 7cc) and stir thoroughly for about one minute. Allow to gel for two minutes.
6. Spread Immediate mixture into the ridge area covering from the crest of the ridge to the periphery and across the post dam area. (When relining a lower denture, all tissue bearing surfaces of the denture should be covered.)
7. Place denture into patient's mouth and have patient close mouth slowly to a biting position.

Instructions for Use (Continued)

Bring teeth to occlusion and have patient suck in cheeks to muscle trim.

8. After three minutes remove denture from patient's mouth to check coverage and muscle trim. Add or trim material, as needed, at this time.

9. Replace denture into patient's mouth and have patient hold a light biting position for about five minutes.

10. Remove denture and trim as needed using a sharp knife or an emery disc.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

Authorized Representative in EU:

MediMark® Europe
BP 2332
38033 Grenoble Cedex 2
France



The LANG logo, featuring the word 'LANG' in a bold, italicized, sans-serif font. A diagonal line extends from the bottom left of the 'L' across the 'A' and 'N'.

Lang Dental Mfg. Co., Inc.
Wheeling, IL 60090-0969
www.LangDental.com
Made in USA
Ph 847-215-6622
Fax 847-215-6678