

# Jet Seal

## Product Identification

Surface sealant for acrylic resin

## Package Contents

REF 4102	Net Vol. 30ml, disposable brushes
REF 4106	Net Vol. 946ml
REF 4107	Net Vol. 3.8L
REF 4108	Net Vol. 19L

## Intended Purpose

This product is intended to be used as an acrylic sealing agent.

## Storage Conditions/Instructions

Close container tightly after use. Store in original container in a cool, dry, well-ventilated location, away from sources of ignition and light.



F - Highly Flammable

## Operating/Handling Instructions

Liquid may cause skin irritation. Avoid contact with skin, eyes and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation. Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at [www.langdental.com](http://www.langdental.com) or by contacting manufacturer.



Xi - Irritant

**Warning:** flammable liquid. Contains: Methyl Methacrylate Monomer, Stabilized. Flash Point: 52.7°F (11.5°C), NFPA, NPCA-HMIS #: H-2,F-3,R-2.

## Contraindications

Jet Seal is contraindicated in patients with allergies to chemicals.

## Other Materials/Equipment Needed

1. Disposable brush

## Instructions for Use

1. Using brush, paint a thin layer of Jet Seal over the area to be sealed. Allow Jet Seal to cure for 1-2 minutes. (Avoid thick layers.)
2. Repeat process to achieve a high shine if desired.

## Disposal

Dispose of unused material in accordance with local, state, and national legislation.

## Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



MediMark<sup>®</sup> Europe  
BP 2332  
38033 Grenoble Cedex 2  
France

The logo features the word "LANG" in a bold, italicized, sans-serif font. A thin horizontal line is positioned above the letters. A diagonal line starts from the bottom left and passes through the letter "A".

*LANG*

Lang Dental Mfg. Co., Inc.  
Wheeling, IL 60090-0969  
[www.LangDental.com](http://www.LangDental.com)  
Made in USA  
Ph 847-215-6622  
Fax 847-215-6678