

Liquid Foil

Product Identification

Separating Solution

Package Contents

REF 2203	Net Vol. 118ml
REF 2206	Net Vol. 1.9L
REF 2208	Net Vol. 5 gal.
REF 2230	Net Wt. 454g
REF 2270	Net Wt. 11.3kg
REF 2280	Net Wt. 45.4kg

Intended Purpose

This product is intended to separate acrylic from stone model.

Storage Conditions/Instructions

Close container tightly after use. Store in a cool, dry location.

Operating/Handling Instructions

Powder may cause skin irritation. Avoid creating dust cloud in handling, transfer, and clean-up. Avoid contact with skin, eyes and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of dust. Use with adequate ventilation. Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at www.langdental.com or by contacting manufacturer.

Contraindications

Liquid Foil is contraindicated in patients with allergies to chemicals.

Other Materials/Equipment Needed

For Liquid

1. Brush

For Powder

1. Scale or measuring spoon
2. 4-oz bottle with cap

Instructions for Use

For Liquid

1. Soak stone model in water until saturated.
2. Remove from water. Dry surface.
3. Paint thin layer uniformly over area of stone model that will come in contact with acrylic.
4. Let dry approximately three minutes.

For Powder

1. Add 1.5 grams (approximately 1/2 teaspoon) of powder to 70ml of warm water.
2. Shake vigorously for one minute, or until powder is fully dissolved.
3. Continue with Instructions "For Liquid" above.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

The logo for LANG, featuring the word "LANG" in a stylized, bold, sans-serif font. A diagonal line passes through the letters, starting from the bottom left of the 'L' and extending towards the top right of the 'G'.

Lang Dental Mfg. Co., Inc.
Wheeling, IL 60090-0969
www.LangDental.com
Made in USA
Ph 847-215-6622
Fax 847-215-6678

LDD 22 REV.3