

Ortho-Jet Tinting Concentrate

Product Identification

Dye for acrylic resin liquid

Package Contents

REF 3302	Net Vol. 59ml
REF 3304	Net Vol. 236ml
REF 3306	Net Vol. 946ml

Intended Purpose

This product is intended for the addition of color to clear orthodontic acrylic liquids.


Storage Conditions/Instructions

Close container tightly after use. Store in original container in a cool, dry, well-ventilated location, away from sources of ignition and light.

Operating/Handling Instructions



F - Highly Flammable

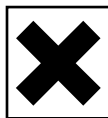

Xn - Harmful

Liquid may cause skin irritation. Avoid contact with skin, eyes and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation. Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at www.langdental.com or by contacting manufacturer.

Warning: flammable liquid. Contains: Methyl Methacrylate Monomer, Inhibited. Flash Point: 52.7°F (11.5°C), NFPA, NPCA-HMIS #: H-2,F-3, R-2.

Contraindications

Ortho-Jet Tinting Concentrate is contraindicated in patients with allergies to chemicals.

Other Materials/Equipment Needed

1. Glass dropper
2. Ortho-Jet liquid - clear

Instructions for Use

For consistent results always shake liquid before use

1. Add a small amount of tinting concentrate to Ortho-Jet clear liquid.
2. Stir until homogenous.
3. Repeat until desired tint is achieved.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is used for other than intended purpose or other than instructed.

Authorized Representative in EU:

MediMark[®] Europe
BP 2332
38033 Grenoble Cedex 2
France



Lang Dental Mfg. Co., Inc.
Wheeling, IL 60090-0969
www.LangDental.com
Made in USA
Ph 847-215-6622
Fax 847-215-6678

The logo for LANG features the word 'LANG' in a bold, italicized, sans-serif font. The letters are underlined with a thin horizontal line. A diagonal line extends from the bottom left of the 'L' across the 'A' and 'N'.