

Orthodontic Glitter

Product Identification

Orthodontic glitter

Package Contents

REF 2410 Net Wt. 28 grams

Intended Purpose

This product is intended as a decorative enhancement to orthodontic appliances.

Storage Conditions/Instructions



Keep away from heat, sparks and open flame. Keep container closed when not in use. Store between 36°F and 90°F (2°C and 32°C).

Operating/Handling Instructions

Always practice good hygiene when using glitter. Avoid contact with hands and mouth. Use safety glasses, dust mask and protective clothing as needed. See Material Safety Data Sheet for complete information. MSDS is available at www.langdental.com or by contacting manufacturer.

Other Materials/Equipment Needed

Ortho-Jet Acrylic

Instructions for Use

1. Following instructions for Ortho-Jet acrylic, sprinkle or mix glitter into acrylic. For best results, ensure that acrylic fully surrounds the glitter.
2. Continue curing as directed.

Disposal


Landfill or incinerate in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is used for other than intended purpose or other than instructed.

Authorized Representative in EU:

MediMark® Europe
BP 2332
38033 Grenoble Cedex 2
France

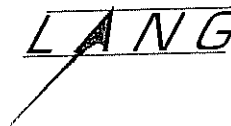

Lang Dental Mfg. Co., Inc.
Wheeling, IL 60090-0969
www.LangDental.com
Made in USA
Ph 847-215-6622
Fax 847-215-6678