

Alginout®

Intended Purpose

Cleaning alginate from impression trays and instruments

Contraindications

Alginout is contraindicated in users with allergies to chemicals.

Other Materials / Equipment Needed

1. Bowl
2. Water

Instructions for Use

1. Mix 1 scoop of Alginout with 16 oz. (473 mL) of water in bowl (1 scoop = approx. 15 cc).
2. Soak until clean.

Handling Precautions

Wear protective gloves, safety glasses with side shields (or goggles) and a face shield. Wash hands thoroughly after handling and before eating. Avoid breathing vapors. Do not eat, drink or smoke in areas where this material is used.

First Aid

IF IN EYES: Immediately flush eyes with water. IF ON SKIN: Wash with soap and water. IF INHALED: Remove from area of exposure. If not breathing, give artificial respiration. If breathing is difficult, give oxygen. Keep victim warm. IF SWALLOWED: Do not induce vomiting. Do not give liquids. Obtain emergency medical attention when necessary.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if not used for intended purpose, not used as instructed or not stored in original container.

Lang Dental Mfg. Co., Inc.
175 Messner Drive
Wheeling, IL 60090 USA
www.LangDental.com

LANG[®]

Made in USA
Ph +1 847-215-6622
Fax +1 847-215-6678



©2020 Lang Dental Mfg. Co., Inc. LANG and ALGINOUT are the registered trademarks of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 34 REV. 3