

# Alginout®

## Product Identification

Impression tray and instrument cleaner

## Package Contents

REF 3430                      Net Wt. 650g; measuring scoop

## Intended Purpose

Cleaning alginate from impression trays and instruments

## Contraindications

Alginout is contraindicated in users with allergies to chemicals.

## Storage

Keep in original container tightly closed. Store in a well-ventilated place.

## Operating / Handling Instructions

Danger. Causes severe skin burns and eye damage. Harmful if swallowed. May cause respiratory irritation. May be corrosive to metals. Wear protective gloves/protective clothing/eye protection/face protection. Wash face, hands and any exposed skin thoroughly after handling. Do not eat, drink or smoke when using this product. Do not breathe dust, fumes, gas, mist, vapors or spray.

## First Aid

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call POISON CENTER or doctor/physician. IF ON SKIN (or hair): Remove/take off immediately all contaminated clothing. Rinse skin with water/shower. Wash contaminated clothing before reuse. IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing. IF SWALLOWED: Rinse mouth. DO NOT induce vomiting. Call a POISON CENTER or doctor/physician if you feel unwell.

## **Other Materials / Equipment Needed**

1. Bowl
2. Water

## **Instructions for Use**

1. Mix 1 scoop of Alginout with 16oz (473mL) of water in bowl (1 scoop = approx. 15cc)
2. Soak until clean

## **Disposal**

Dispose of unused material in accordance with local, state, and national legislation.

## **Warranty**

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if not used for intended purpose, not used as instructed or not stored in original container.

Lang Dental Mfg. Co., Inc.  
175 Messner Drive  
Wheeling, IL 60090 USA  
[www.LangDental.com](http://www.LangDental.com)

The logo for LANG features the word "LANG" in a bold, italicized, sans-serif font. A horizontal line is positioned above the letters, and a diagonal slash cuts through the line from the bottom left to the top right, passing behind the letter "A". A registered trademark symbol (®) is located to the upper right of the "G".

Made in USA  
Ph 1-847-215-6622  
Fax 1-847-215-6678



©2016 Lang Dental Mfg. Co., Inc. LANG and ALGINOUT are the registered trademarks of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 34 REV. 2