

El Cor®

Intended Purpose

Removal of zinc oxide eugenol cements and pastes

Other Materials/Equipment Needed

Disposable cloth

Instructions for Use

Note: If cleaning plastic, test El Cor® on small patch of surface before using. Avoid use on vinyl.

1. Apply small amount to disposable cloth.
2. Wipe until clean.

Handling Precautions

Contains orange terpenes.

Flammable liquid and vapor.

Keep away from heat/sparks/open flames/hot surface. – No smoking.

Causes skin irritation. May cause an allergic skin reaction.

Wear protective gloves/protective clothing/eye protection/face protection.

Keep material in original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

Disposal

Dispose of contents and container in accordance with all local, regional and national regulations.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase only from an authorized reseller. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



Lang Dental Mfg. Co., Inc.

175 Messner Dr.

Wheeling, IL 60090 USA

www.langdental.com

Made in USA

Ph. +1 847-215-6622

Fax +1 847-215-6678

© 2021 Lang Dental Mfg. Co., Inc. LANG and EL COR are the registered trademarks of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 04 R.1