

Jet™ Tooth Shade Acrylic Resin

Intended Purpose

Fabrication, by a dental professional, of fast-curing provisional crowns and bridges intended to remain in the mouth for less than 30 days. **CONTRAINDICATION:** Persons with sensitivity to methyl methacrylate monomer or methyl methacrylate polymer.

Other Materials / Equipment Needed

1. Silicone lubricant
2. Acrylic bur or stone
3. Aquapres® (pressure curing unit), optional
4. Jet™ Adjusters (acrylic stains), optional
5. Jet™ Seal (acrylic sealant), optional

Instructions for Use

For consistent results always shake Jet™ liquid before use, as catalyst may have separated from monomer.

1. In the impression, fill the tooth or teeth that were extracted or prepared with Jet™ Tooth Shade as follows. Moisten surface with Jet™ liquid. Spray on Jet™ Tooth Shade powder from dispenser bottle. Alternate liquid and powder until impression is filled. For best results, begin and end with liquid.
2. While allowing acrylic to reach a paste like consistency, coat the prepared teeth and surrounding tissue with silicone lubricant to allow easy removal of acrylic from prep and to protect the teeth and surrounding tissue.
3. Seat impression in mouth and hold in place for 3 minutes. Remove and trim excess acrylic. Do not over trim.
4. Reseat impression in mouth and place in and out of patient's mouth until material reaches a rubbery state and cannot be distorted. **CAUTION!** Do not allow material to get hot in mouth!

5. Remove impression from patient's mouth. Allow an additional 10 minutes for curing, either on bench, or for better results, in Aquapres® at 30 psi.
6. Remove provisional from impression and check for fit.
7. Trim excess material with an acrylic bur or stone. Grind to a smooth finish.
8. Modify shade with Jet™ Adjusters if desired. Polish using Jet™ Seal.
9. Clean patient's teeth of any residual silicone lube. Place provisional into mouth and secure with suitable temporary cement.

Storage

Keep original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

Operating/Handling Instructions

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surfaces – No smoking. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Avoid breathing vapors. Wear protective gloves/protective clothing/eye protection/face protection.

First Aid

If inhaled: Remove person to fresh air and keep comfortable for breathing. Call a Poison Center or doctor if you feel unwell. If on skin: Wash with plenty of water and soap. If skin irritation or rash occurs: Get medical advice. Take off contaminated clothing and wash it before use. If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Get immediate medical advice. If swallowed: Rinse mouth. Do NOT induce vomiting. Get immediate medical advice. See Safety Data Sheet for complete information.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

Package Contents

REF 1410	Net Wt. 45 g
REF 1420	Net Wt. 100 g
REF 1430	Net Wt. 454 g
REF 1450	Net Wt. 2.3 kg
REF 1470	Net Wt. 11.3 kg
REF 1480	Net Wt. 45.4 kg
REF 1402X6	Net Vol. 6x30 mL
REF 1403	Net Vol. 118 mL
REF 1404	Net Vol. 236 mL
REF 1405	Net Vol. 473 mL
REF 1406	Net Vol. 946 mL
REF 1407	Net Vol. 3.8 L
REF 1408	Net Vol. 19 L
REF 1407D	Net Vol. 202.69 L
REF 1484	Net Wt. 4x45 g; Net Vol. 118 mL; glass dropper
REF 1493	Net Wt. 6x45 g; Net Vol. 118 mL; glass dropper



MediMark[®] Europe Sarl
BP 2332
38033 Grenoble Cedex 2
France



Lang Dental Mfg. Co., Inc.
175 Messner Dr.
Wheeling, IL 60090 USA
www.LangDental.com



Made in USA
Ph +1-847-215-6622
Fax +1-847-215-6678



Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2018 Lang Dental Mfg. Co., Inc. LANG and AQUAPRES are the registered trademarks of Lang Dental Mfg. Co., Inc. JET is the trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 14 R.13 2018-06