

# Immediate™ Acrylic Resin

## Intended Purpose

Fabrication, by a dental professional, of denture relines

## Other Materials / Equipment Needed

1. Acrylic bur or ruby carver
2. Acrylic Primer (Acrylic bonding agent)
3. Brush
4. Spatula
5. Sharp knife or emery disc



## Instructions for Use

For consistent results always shake liquid before use.

1. Clean and disinfect denture.
2. Trim down any high spots and relieve denture if necessary with acrylic burs or ruby carvers.
3. Coat denture area to be relined with Acrylic Primer.
4. Protect any surfaces not to be relined.
5. Mix approximately 2 parts Immediate™ powder with 1 part Immediate™ Liquid. Stir thoroughly for about one minute. Allow to gel for two minutes.
6. Spread mixture into the ridge area covering from the crest of the ridge to the periphery and across the post dam area. (When relining a lower denture, all tissue bearing surfaces of the denture should be covered.)
7. Place denture into patient's mouth and have patient close mouth slowly to a biting position. Bring teeth to occlusion and have patient suck in cheeks to muscle trim.

8. After three minutes remove denture from patient's mouth to check coverage and muscle trim. Add or trim material as needed.

9. Replace denture into patient's mouth and have patient hold a light biting position for about five minutes.

10. Remove denture and trim as needed using a sharp knife or an emery disc.

### **Handling Precautions**

Contains methyl methacrylate

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surface. – No smoking.

Causes skin irritation. May cause an allergic skin reaction.

May cause respiratory irritation.

Wear protective gloves/protective clothing/eye protection/face protection.

Keep material in original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

### **Disposal**

Dispose of contents and container in accordance with all local, regional and national regulations.

### **Warranty**

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase only from an authorized reseller. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

Lang Dental Mfg. Co., Inc.  
175 Messner Dr  
Wheeling, IL 60090 USA  
www.langdental.com  
Ph. +1 847-215-6622  
Fax +1 847-215-6678  
Made in USA

*LANG*<sup>®</sup>



MediMark<sup>®</sup> Europe  
11 rue E. Zola  
38100 Grenoble  
France



Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2021 Lang Dental Mfg. Co., Inc. LANG is the registered trademark of Lang Dental Mfg. Co., Inc. IMMEDIATE is the trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 37 R. 6 2021-07

