

# Tempo<sup>®</sup> Acrylic Resin

## Intended Purpose

Fabrication, by a dental professional, of denture relines

## Other Materials / Equipment Needed

1. Spatula
2. Knife



## Instructions for Use

For consistent results always shake Tempo<sup>®</sup> liquid before use.

## Soft Denture Liner

1. Clean and disinfect denture base. Grind down areas that require relief.
2. Mix approximately 2 parts Tempo<sup>®</sup> powder with 1 part Tempo<sup>®</sup> liquid, stirring thoroughly for 10 seconds.
3. Allow mixture to gel until it becomes viscous, yet flowable. Pour into denture areas requiring treatment.
4. Have patient rinse mouth with water. Insert denture into patient's mouth. Have patient come to a bite rest position to align occlusion and hold this position for three minutes. Patient can then talk, read aloud or chew gently to generate a functional impression. Total curing time is 20 minutes from initial mixing.
5. Remove relined denture from patient's mouth. Check to see if peripheral border is adequately covered and sufficient material has been used to give treatment area full coverage. Additional Tempo<sup>®</sup> can be added at any time. Trim excess material with a sharp knife.
6. Have patient return periodically to note progress of treatment. To fix sore spots occurring as a result of denture base showing through reline, grind down reline and add another coat of Tempo<sup>®</sup>.

**NOTE:** Tempo<sup>®</sup> can be removed from a denture base by scraping with a knife. The thin surface layer can then be dissolved with methyl methacrylate based acrylic monomer.

## **Functional Impression Material**

To use Tempo® as a functional impression material only, follow steps 1-5 above. Allow patient to wear denture with Tempo® for 5 to 24 hours. Pour up stone model as soon as possible after removal of impression from the patient's mouth.

## **Handling Precautions**

Contains ethyl alcohol

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surface. – No smoking. May cause an allergic skin reaction. Causes serious eye irritation. Wear protective gloves/protective clothing/eye protection/face protection. Keep material in original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

## **Disposal**

Dispose of contents and container in accordance with all local, regional, national and international regulations.

## **Warranty**

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase only from an authorized reseller. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

Lang Dental Mfg. Co., Inc.  
175 Messner Dr.  
Wheeling, IL 60090 USA  
www.langdental.com  
Ph. +1 847-215-6622  
Fax +1 847-215-6678  
Made in USA

*LANG*<sup>®</sup>



MediMark® Europe  
11 rue E. Zola  
38100 Grenoble  
France

Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2020 Lang Dental Mfg. Co., Inc. LANG and TEMPO are the registered trademarks of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 10 R. 10 2020-10

