

Jet™ Adjusters Liquid

Intended Purpose

Color modification, by a dental professional, of provisional crowns

Other Materials / Equipment Needed

1. Disposable Brush

Instructions for Use



For consistent results always shake Jet™ Adjusters before use.

Note: Jet™ Adjuster Thinner can be added to Jet™ Adjuster colors to reduce viscosity if the liquid becomes too thick.

1. Finish case as usual. Do not polish.
2. Dip brush into Jet™ Adjuster and apply thin coat to desired area of crown. Allow application to dry at least 30 seconds.
3. Apply subsequent coats as needed to achieve more color intensity.
4. Clean brush in Jet™ Adjuster Brush Cleaner before changing colors or using Jet™ Adjuster Thinner.
5. After staining is complete, apply Jet™ Adjuster Clear over surface for a lustrous shine.
6. Allow Jet™ Adjusters to cure for at least 2 minutes before cementing restoration into mouth.
7. Clean top of bottle before replacing cap.

Suggested Applications:

Blue: For incisal translucency and to neutralize an orange shade.

Brown: To darken gingival area and to characterize stain of hairline cracks and tobacco stains.

Grey: For incisal translucency and to decrease intensity of any shade.

Orange: To darken gingival area and to characterize stain of hairline cracks and tobacco stains.

Violet: To make shades more translucent and to neutralize a yellow shade.

White: For characterization stain of calcification spots and incisal area, and to lighten shade.

Yellow: To change overall color of tooth.

To Darken Tooth Shades:	Use:
59, 60, 61, 62 & 66	Brown or orange
65 & 67	Grey or yellow
69, 77, 81 & 87	Brown or grey

Handling Precautions

Contains methyl methacrylate

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surface. – No smoking.

Causes skin irritation. May cause an allergic skin reaction.

May cause respiratory irritation.

Wear protective gloves/protective clothing/eye protection/face protection.

Keep material in original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

Disposal

Dispose of contents and container in accordance with all local, regional, and national regulations.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase from an authorized dealer. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



MediMark® Europe. 11 rue E. Zola
38100 Grenoble. France



Lang Dental Mfg. Co., Inc.
175 Messner Dr.
Wheeling, IL 60090 USA
www.LangDental.com



Made in USA
Ph +1 847-215-6622
Fax +1 847-215-6678



Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2021 Lang Dental Mfg. Co., Inc. LANG is the registered trademark of Lang Dental Mfg. Co., Inc. JET is the trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 32 R. 8 2021-02

