

Jet™ Seal

Intended Purpose

Sealing of acrylic surfaces by a dental professional

Other Materials / Equipment Needed

1. Disposable Brush

Instructions for Use

1. Using brush, paint a thin layer of Jet™ Seal over the area to be sealed. Allow Jet™ Seal to cure for 1-2 minutes. (Avoid thick layers.)
2. Repeat process to achieve a high shine if desired.

Handling Precautions

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surfaces – No smoking. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Avoid breathing vapors. Wear protective gloves/protective clothing/eye protection/face protection.

Disposal

Dispose of contents and container in accordance with all local, regional and national regulations.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase only from an authorized reseller. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

Lang Dental Mfg. Co., Inc.
175 Messner Dr
Wheeling, IL 60090 USA
www.langdental.com
Ph. +1 847-215-6622
Fax +1 847-215-6678
Made in USA

LANG[®]



MediMark[®] Europe
11 rue E. Zola
38100 Grenoble
France



Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2021 Lang Dental Mfg. Co., Inc. LANG is the registered trademark of Lang Dental Mfg. Co., Inc. Jet is the trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 41 R.12 2021-07