

# Jet™ Adjusters Liquid

## Product Identification

Resin fluid for modifying esthetic appearance of provisional crowns

## Package Contents

REF 3201	Net Vol. 10mL
REF 3206	Net Vol. 946mL
REF 3299	Net Vol. 10x10mL; disposable brushes

## Intended Purpose

Color modification of provisional crowns and bridges

## Contraindications

Jet Adjusters is contraindicated for persons with sensitivity to methyl methacrylate.

## Storage

Keep original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

## Operating / Handling Instructions

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surfaces – No smoking. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Avoid breathing vapors. Wear protective gloves/protective clothing/eye protection/face protection.

## First Aid

If inhaled: Remove person to fresh air and keep comfortable for breathing. Call a Poison Center or doctor if you feel unwell. If on skin: Wash with plenty of water and soap. If skin irritation or rash occurs: Get medical advice. Take off contaminated clothing and wash it before use. If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Get immediate medical advice. If swallowed: Rinse mouth. Do NOT induce vomiting. Get immediate medical advice.

## **Other Materials / Equipment Needed**

1. Disposable Brush

## **Instructions for Use**

Note: Jet Adjuster Thinner can be added to Jet Adjuster colors to reduce viscosity if the liquid becomes too thick.

1. Finish case as usual. Do not polish.
2. Shake Jet Adjuster liquid before using. Dip brush into Jet Adjuster and apply thin coat to desired area of tooth. Allow application to dry at least 30 seconds.
3. Apply subsequent coats as needed to achieve more color intensity.
4. Clean brush in Jet Adjuster Brush Cleaner before changing colors or using Jet Adjuster Thinner.
5. After staining is complete, apply Jet Adjuster Clear over surface for a lustrous shine.
6. Allow Jet Adjusters to cure for at least 2 minutes before cementing restoration into mouth.
7. Clean top of bottle before replacing cap.

## **Suggested Applications:**

**Blue:** For incisal translucency and to neutralize an orange shade.

**Brown:** To darken gingival area and to characterize stain of hairline cracks and tobacco stains.

**Grey:** For incisal translucency and to decrease intensity of any shade.

**Orange:** To darken gingival area and to characterize stain of hairline cracks and tobacco stains.

**Violet:** To make shades more translucent and to neutralize a yellow shade.

**White:** For characterization stain of calcification spots and incisal area, and to lighten shade.

**Yellow:** To change overall color of tooth.

**To Darken Tooth Shades:**

59, 60, 61, 62 & 66  
65 & 67  
69, 77, 81 & 87

**Use:**

Brown or orange  
Grey or yellow  
Brown or grey

**Disposal**

Dispose of unused material in accordance with local, state, and national legislation.

**Warranty**

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



MediMark® Europe Sarl  
BP 2332  
38033 Grenoble Cedex 2  
France



Lang Dental Mfg. Co., Inc.  
175 Messner Drive  
Wheeling, IL 60090 USA  
[www.LangDental.com](http://www.LangDental.com)  
Made in USA  
Ph 1-847-215-6622  
Fax 1-847-215-6678



© 2017 Lang Dental Mfg. Co., Inc. LANG is the registered trademark of Lang Dental Mfg. Co., Inc. JET is the trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 32 R.7

