

# Liquid Foil

## Product Identification

Separating Solution

## Package Contents

REF 2203	Net Vol. 118 mL
REF 2206	Net Vol. 1.9 L
REF 2208	Net Vol. 5 gallon
REF 2230	Net Wt. 454 g
REF 2270	Net Wt. 11.3 kg
REF 2280	Net Wt. 45.4 kg

## Intended Purpose

To facilitate the separation of acrylic from stone model

## Contraindications

Liquid Foil is contraindicated for persons with sensitivity to chemicals.

## Storage

Close container tightly after use. Store in a cool, dry location.

## Operating / Handling Instructions

Handle in accordance with good industrial hygiene and safety practice. Use only in well-ventilated areas. Avoid contact with skin, eyes or clothing. Avoid breathing dust. Wash thoroughly with soap and water after handling. Wash contaminated clothing before reuse. Minimize dust generation and accumulation. See Safety Data Sheet for complete information. SDS is available at [www.langdental.com](http://www.langdental.com) or by contacting manufacturer.

## Other Materials / Equipment Needed

### For Liquid

1. Brush

### For Powder

1. Scale or measuring spoon
2. 4-oz. bottle with cap

## **Instructions for Use**

### **For Liquid**

1. Soak stone model in water until saturated.
2. Remove from water. Dry surface.
3. Paint thin layer uniformly over area of stone model that will come in contact with acrylic.
4. Let dry approximately three minutes.

### **For Powder**

1. Add 1.5 grams (approximately 1/2 teaspoon) of powder to 70 mL of warm water.
2. Shake vigorously for one minute, or until powder is fully dissolved.
3. Continue with Instructions "For Liquid" above.

### **Disposal**

Dispose of unused material in accordance with local, state, and national legislation.

### **Warranty**

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

Lang Dental Mfg. Co., Inc.  
175 Messner Dr.  
Wheeling, IL 60090 USA  
[www.LangDental.com](http://www.LangDental.com)  
Made in USA  
Ph 1-847-215-6622  
Fax 1-847-215-6678



© 2018 Lang Dental Mfg. Co., Inc. LANG is a registered trademark of Lang Dental Mfg. Co., Inc. All rights reserved.