

Ortho-Jet™ BCA

Product Identification

Self Curing Acrylic Resin

Package Contents

| | |
|-----------|--|
| REF B1320 | Net Wt. 100g |
| REF B1330 | Net Wt. 454g |
| REF B1350 | Net Wt. 2.3kg |
| REF B1303 | Net Vol. 118ml |
| REF B1304 | Net Vol. 236ml |
| REF B1306 | Net Vol. 946ml |
| REF B1323 | Net Wt. 100g; Net Vol. 118ml; glass dropper |
| REF B1334 | Net Wt. 454g; Net Vol. 236ml; glass dropper; dispenser bottle |
| REF B1356 | Net Wt. 2.3kg; Net Vol. 946ml |

Intended Purpose

This product is intended for research use in potting, embedding and cementation.

Storage Conditions/Instructions

Close container tightly after use. Store in original container, in a cool, dry, well-ventilated location, away from sources of ignition and light.



F - Highly Flammable

Operating/Handling Instructions

Liquid may cause skin irritation. Avoid contact with skin, eyes and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation. Do not smoke in the presence of open containers. See Material Safety Data Sheet for complete information. MSDS is available at www.langdental.com or by contacting manufacturer.



Xn - Harmful

Warning: flammable liquid. Contains: Methyl Methacrylate Monomer, stabilized. Flash Point: 52.7°F (11.5°C), NFPA, NPCA-HMIS #: H-2,F-3,R-2.

Other Materials Needed

1. Mixing cup
2. Stirring instrument

Instructions for Use

For consistent results always shake liquid before use.

1. Mix 2 parts by volume of Ortho-Jet BCA powder with 1 part by volume of Ortho-Jet BCA liquid. (For lower viscosity, mix 3 parts by volume of powder with 2 parts liquid).
2. Blend mixture thoroughly for 15-20 seconds.
3. Apply mixture as needed.
4. Allow to cure at room temperature.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purposes, not used as instructed or not stored in original container.

Lang Dental Mfg. Co., Inc.
Wheeling, IL 60090-0969
www.LangDental.com
Made in USA
Ph 847-215-6622
Fax 847-215-6678

The logo for Lang Dental features the word "LANG" in a bold, italicized, sans-serif font. The letters are contained within a thin horizontal rectangular border. A diagonal line extends from the bottom left corner of the border, passing through the letter 'A' and extending towards the bottom left of the page.