

Ortho-Jet™ Tinting Concentrate

Product Identification

Dye for acrylic resin liquid

Package Contents

REF 3302	Net Vol. 59mL
REF 3304	Net Vol. 236mL
REF 3306	Net Vol. 946mL
REF 3307	Net Vol. 3.8L

Intended Purpose

This product is intended for the addition of color to clear orthodontic liquids.

Storage Conditions / Instructions

Close container tightly after use. Store in original container in a cool, dry, well-ventilated location, away from sources of ignition and light.

Operating / Handling Instructions



Xn - Harmful

Liquid may cause skin irritation. Avoid contact with skin, eyes and clothing. Wear protective gloves, safety glasses with side shields or chemical splash goggles. Avoid prolonged breathing of vapors. Use with adequate ventilation. Do not smoke in the presence of open containers. See Safety Data Sheet for complete information. SDS is available at www.langdental.com or by contacting manufacturer.



F - Highly Flammable

Warning: flammable liquid. Contains: Methyl Methacrylate Monomer, stabilized. Flash Point: 52.7°F (11.5°C), NFPA, NPCA-HMIS #: H-2, F-3, R-2.

Contraindications

Ortho-Jet Tinting Concentrate is contraindicated for persons with allergies to methyl methacrylate monomer.

Other Materials / Equipment Needed

1. Glass dropper
2. Ortho-Jet liquid - clear

Instructions for Use

For consistent results always shake liquids before use, as catalyst may have separated from monomer.

1. Add a small amount of Ortho-Jet Tinting Concentrate to Ortho-Jet clear liquid.
2. Stir until homogenous.
3. Repeat until desired tint is achieved.

Disposal

Dispose of unused material in accordance with local, state and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from the date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



MediMark[®] Europe Sarl
BP 2332
38033 Grenoble Cedex 2
France

Lang Dental Mfg. Co., Inc.
175 Messner Dr.
Wheeling, IL 60090 USA
www.LangDental.com

Made in USA
Ph 847-215-6622
Fax 847-215-6678

The logo for LANG, featuring the word 'LANG' in a bold, italicized, sans-serif font. A thin horizontal line is positioned above the letters, and a diagonal line extends from the bottom left of the 'L' towards the top right of the 'G'.

LANG[®]



© 2013 Lang Dental Mfg. Co., Inc. LANG is a registered trademark of Lang Dental Mfg. Co., Inc. ORTHO-JET is a trademark of Lang Dental Mfg. Co., Inc. All rights reserved.