

Tempo[®] Acrylic Resin

Intended Purpose

Fabrication, by a dental professional, of denture relines.

CONTRAINDICATION: Persons with sensitivity to denatured ethanol.

Other Materials / Equipment Needed

1. Spatula
2. Knife

Instructions for Use

For consistent results, always shake Tempo[®] liquid before use.

A. Soft Denture Liner

1. Clean and disinfect denture base. Grind down areas that require relief.
2. Mix approximately 2 parts Tempo[®] powder with 1 part Tempo[®] liquid, stirring thoroughly for 10 seconds.
3. Allow mixture to gel until it becomes viscous, yet flowable. Pour into denture areas requiring treatment.
4. Have patient rinse mouth with water. Insert denture into patient's mouth. Have patient come to a bite rest position to align occlusion and hold this position for three minutes. Patient can then talk, read aloud or chew gently to generate a functional impression. Total curing time is 20 minutes from initial mixing.
5. Remove relined denture from patient's mouth. Check to see if peripheral border is adequately covered and sufficient material has been used to give treatment area full coverage. Additional Tempo[®] can be added at any time. Trim excess material with a sharp knife.
6. Have patient return periodically to note progress of treatment. To fix sore spots occurring as a result of denture base showing through reline, grind down reline and add another coat of Tempo[®].

NOTE: Tempo® can be removed from a denture base by scraping with a knife. The thin surface layer can then be dissolved with methyl methacrylate based acrylic monomer.

B. Functional Impression Material

To use Tempo® as a functional impression material only, follow steps 1-5. Allow patient to wear denture with Tempo® for 5 to 24 hours. Pour up stone model as soon as possible after removal of impression from the patient's mouth.

Storage

Keep original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

Operating/Handling Instructions

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surfaces – No smoking. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Avoid breathing vapors. Wear protective gloves/protective clothing/eye protection/face protection.

First Aid

If inhaled: Remove person to fresh air and keep comfortable for breathing. Call a Poison Center or doctor if you feel unwell. If on skin: Wash with plenty of water and soap. If skin irritation or rash occurs: Get medical advice. Take off contaminated clothing and wash it before use. If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Get immediate medical advice. If swallowed: Rinse mouth. Do NOT induce vomiting. Get immediate medical advice. See Safety Data Sheet for complete information.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.

Package Contents

REF 1020	Net Wt. 100 g
REF 1030	Net Wt. 454 g
REF 1050	Net Wt. 2.7 kg
REF 1070	Net Wt. 11.3 kg
REF 1080	Net Wt. 45.4 kg
REF 1003	Net Vol. 118 mL
REF 1004	Net Vol. 236 mL
REF 1006	Net Vol. 946 mL
REF 1007	Net Vol. 3.8 L
REF 1008	Net Vol. 19 L
REF 1009L	Net Vol. 200 L
REF 1023	Net Wt. 100 g; Net Vol. 118 mL; mixing cups
REF 1034	Net Wt. 454 g; Net Vol. 236 mL; mixing cups
REF 1056	Net Wt. 2.7 kg; Net Vol. 946 mL; mixing cups



MediMark® Europe Sarl
BP 2332
38033 Grenoble Cedex 2
France



Lang Dental Mfg. Co., Inc.
175 Messner Dr.
Wheeling, IL 60090 USA
www.LangDental.com



Made in USA
Ph +1-847-215-6622
Fax +1-847-215-6678



Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2018 Lang Dental Mfg. Co., Inc. Tempo and LANG are the registered trademarks of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 10 REV. 9 2018-07