

Immediate™ Acrylic Resin

Product Identification

Very soft denture reline acrylic resin

Package Contents

REF 3720	Net Wt. 100 g
REF 3730	Net Wt. 454 g
REF 3750	Net Wt. 2.7 kg
REF 3770	Net Wt. 11.3 kg
REF 3780	Net Wt. 45.4 kg
REF 3703	Net Vol. 118 mL
REF 3704	Net Vol. 236 mL
REF 3706	Net Vol. 946 mL
REF 3707	Net Vol. 3.8 L
REF 3708	Net Vol. 19 L
REF 3723	Net Wt. 100 g ; Net Vol. 118 mL; Acrylic Primer: Net Vol. 10 mL; mixing cups, brushes
REF 3734	Net Wt. 454 g; Net Vol. 236 mL; Acrylic Primer: Net Vol. 10 mL; mixing cups, brushes
REF 3756	Net Wt. 2.7 kg; Net Vol. 946 mL Acrylic Primer: Net Vol. 10 mL; mixing cups, brushes

Intended Purpose

Fabrication of denture reline; Target Group: persons who wear dentures; for prescription use only

Contraindications

Immediate™ is contraindicated for persons with sensitivity to methyl methacrylate monomer.

Storage

Keep original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

Operating / Handling Instructions

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surfaces – No smoking. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Avoid breathing vapors. Wear protective gloves/protective clothing/eye protection/face protection.

First Aid

If inhaled: Remove person to fresh air and keep comfortable for breathing. Call a Poison Center or doctor if you feel unwell. If on skin: Wash with plenty of water and soap. If skin irritation or rash occurs: Get medical advice. Take off contaminated clothing and wash it before use. If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Get immediate medical advice. If swallowed: Rinse mouth. Do NOT induce vomiting. Get immediate medical advice. See Safety Data Sheet for complete safety information.

Other Materials / Equipment Needed

1. Acrylic bur or ruby carver
2. Acrylic Primer (Acrylic bonding agent)
3. Brush
4. Spatula
5. Sharp knife or emery disc

Instructions for Use

For consistent results always shake Immediate™ liquid before use, as catalyst may have separated from monomer.

1. Clean and disinfect denture.
2. Trim down any high spots and relieve denture if necessary with acrylic burs or ruby carvers.
3. Coat denture area to be relined with Acrylic Primer.

4. Protect any surfaces not to be relined.
5. Mix approximately 2 parts Immediate™ powder with 1 part Immediate™ Liquid. Stir thoroughly for about one minute. Allow to gel for two minutes.
6. Spread mixture into the ridge area covering from the crest of the ridge to the periphery and across the post dam area. (When relining a lower denture, all tissue bearing surfaces of the denture should be covered.)
7. Place denture into patient's mouth and have patient close mouth slowly to a biting position. Bring teeth to occlusion and have patient suck in cheeks to muscle trim.
8. After three minutes remove denture from patient's mouth to check coverage and muscle trim. Add or trim material as needed.
9. Replace denture into patient's mouth and have patient hold a light biting position for about five minutes.
10. Remove denture and trim as needed using a sharp knife or an emery disc.

Disposal

Dispose of unused material in accordance with local, state and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from the date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



MediMark[®] Europe Sarl
BP 2332
38033 Grenoble Cedex 2
France



Lang Dental Mfg. Co., Inc.
175 Messner Dr.
Wheeling, IL 60090 USA
www.LangDental.com



Made in USA
Ph 1-847-215-6622
Fax 1-847-215-6678



Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2018 Lang Dental Mfg. Co., Inc. LANG is the registered trademark of Lang Dental Mfg. Co., Inc. IMMEDIATE is the trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 37 R. 5 2018-05