

Jet™ Seal

Intended Purpose

Sealing of acrylic surfaces by a dental professional.

CONTRAINDICATIONS: Persons with sensitivity to methyl methacrylate monomer.

Other Materials / Equipment Needed

1. Disposable Brush

Instructions for Use

1. Using brush, paint a thin layer of Jet™ Seal over the area to be sealed. Allow Jet™ Seal to cure for 1-2 minutes. (Avoid thick layers.)
2. Repeat process to achieve a high shine if desired.

Storage

Keep original container tightly closed in upright position. Store in a well-ventilated place. Keep cool.

Operating / Handling Instructions

Danger. Highly flammable liquid and vapor. Keep away from heat/sparks/open flames/hot surfaces – No smoking. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Avoid breathing vapors. Wear protective gloves/protective clothing/eye protection/face protection.

First Aid

If inhaled: Remove person to fresh air and keep comfortable for breathing. Call a Poison Center or doctor if you feel unwell. If on skin: Wash with plenty of water and soap. If skin irritation or rash occurs: Get medical advice. Take off contaminated clothing and wash it before use. If in eyes: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Get immediate medical advice. If swallowed: Rinse mouth. Do NOT induce vomiting. Get immediate medical advice. See Safety Data Sheet for complete information.

Disposal

Dispose of unused material in accordance with local, state, and national legislation.

Warranty

Manufacturer warrants this product to be free from manufacturing defects for one year from date of purchase. Warranty is void if product is not used for intended purpose, not used as instructed or not stored in original container.



MediMark[®] Europe Sarl
BP 2332
38033 Grenoble Cedex 2
France



Lang Dental Mfg. Co., Inc.
175 Messner Drive
Wheeling, IL 60090 USA
www.LangDental.com
Made in USA
Ph +1 847-215-6622
Fax +1 847-215-6678



Note: Any serious incident that has occurred in relation to this device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

© 2019 Lang Dental Mfg. Co., Inc. LANG is the registered trademark of Lang Dental Mfg. Co., Inc. Jet is the trademark of Lang Dental Mfg. Co., Inc. All rights reserved.

LDD 41 R.11 2019-09